

ADD ON RESTRICTIONS FOR COMMON REQUESTS

If test is not listed, refer to Lab User's Guide

* If sample was frozen within 4 hours of receipt, this test may be added to the frozen sample if it has not been thawed (up to 1 month).

ACETAMINOPHEN	Must be a red top	GASTRIN	No add-ons	VALPROIC ACID	Must be a red top
ABG	No add-ons	GENTAMICIN	Must be a red top	VBG	No add-ons
ACTH	No add-ons	GLUCAGON	No add-ons	VIP	No add-ons
ADH	No add-ons	HBVDNA QT	No add-ons	VIT A	No add-ons
APC RESIST	No add-ons	HCVRNQ QT	No add-ons	VIT B1	No add-ons
APTT *	Sample cannot be >4 hours old	HIVRNA QT	No add-ons	VIT B6	No add-ons
ANTIPHOS	No add-ons	HMCYS	Sample cannot be >6 hours old	VIT C	No add-ons
APTT INHIB *	No add-ons	IMM COMP	No add-ons	VOL	No add-ons
AT3 ACT	Sample cannot be >4 hours old	INTACT PTH	Sample cannot be >8 hours old	VW ASSAY	No add-ons
AUTODIF	Sample cannot be >12 hours old	ION CA	No add-ons	VW CLEAV	No add-ons
BNP	Sample cannot be > 24 hours old	LACT	No add-ons	VW MULT	No add-ons
C1Q BIND	No add-ons	LITHIUM	Must be a red top	VW PROFILE	No add-ons
C-AMP	No add-ons	LMW HEPARIN *	Sample cannot be >4 hours old	VWF:AG	No add-ons
CARBAMENZAPINE	Must be a red top	LUPUS COAG	No add-ons	ZINC	No add-ons
CARNITINE	No add-ons	NH3	No add-ons		
CAROTENE	No add-ons	PAI ACT	No add-ons		
CATECH FR	No add-ons	PHENOBARB	Must be a red top		
CBC	Sample cannot be >24 hours old	PHENYTOIN (DIL)	Must be a red top		
CMVDNA QT	No add-ons	PLASMIN	No add-ons		
CYANIDE	No add-ons	PREALBUMIN	Must be an SST or red top		
D-DIMER *	Sample cannot be > 24 hours old	PRO C ACT *	Sample cannot be >4 hours old		
DIFFERENTIAL	Sample cannot be >12 hours old	PRO C ANT	Sample cannot be >12 hours old		
DIGOXIN	Must be a red top	PRO S ACT *	Sample cannot be >4 hours old		
ESR	Sample cannot be > 24 hours old	PRO S ANT	Sample cannot be >12 hours old		
ETHANOL	Up to 24 hours if capped and refrigerated	PSA FREE	Must be an SST or red top		
FIBRINOGEN *	Sample cannot be > 24 hours old	PSA TOTAL	Must be an SST or red top		
F2 ACT *	Sample cannot be >4 hours old	PT *	Sample cannot be > 24 hours old		
F5 ACT *	Sample cannot be >4 hours old	PTH-RP	No add-ons		
F7 ACT *	Sample cannot be >4 hours old	PT INHIBITOR	No add-ons		
F8 ACT *	Sample cannot be >4 hours old	RBC FOLATE	Sample cannot be >24 hours old		
F8 BETHESDA INH	No add-ons	RENIN	No add-ons		
F9 ACT *	Sample cannot be >4 hours old	RETIC COUNT	Sample cannot be >16 hours old		
F9 BETHESDA INH	No add-ons	SALICYLATE	Must be a red top		
F10 ACT *	Sample cannot be >4 hours old	SEROTONIN	No add-ons		
F11 ACT	No add-ons	TT *	Sample cannot be >4 hours old		
F12 ACT	No add-ons	URINALYSIS	No add-ons		
F13 ACT	No add-ons	URINE EOS	Sample cannot be > 24 hours old		
FSH	Must be an SST or red top	VANCOMYCIN	Must be a red top		